EASYBONE QUATTRO

- ✓ **Micromotore chirurgico per implantologia
Physiodispenser**
- ✓ **Surgical micromotor for implant
Physiodispenser**

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LED Light high efficiency,

- ✓ higher visibility of the treated area
- ✓ higher precision

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PROFESSIONALITY AND EFFICIENCY

- ✓ BRUSHLESS MICROMOTOR with 200 to 40.000 rpm speed range: simple and practical functioning at every speed/torque option, and selected program;
- ✓ MICROMOTOR with contra-angle anti-rotation device. Stainless steel motor case and connection cable can be both subject to several auto-cleavable;
- ✓ TORQUE FROM 01 TO 69 NCM with 20:1 contra-angle;
- ✓ SPEED with contra-angle 20:1 from 10 to 2000 rpm;
- ✓ 33 SETTABLE REDUCTIONS;
- ✓ PUMP IRRIGATION LIQUID FLOW adjustable from 0 to 100%;
- ✓ DIRECT ACCESS PROGRAMMABLE FUNCTIONS: speed, torque, pump, reverse, reducer hand-piece (all parameters in sequential mode);
- ✓ ERROS -DIAGNOSTIC SYSTEM CONTR;
- ✓ ANGLE AUTO CALIBRATION upon machine's activation;
- ✓ DIRECT SWITCH FOR EVENTUAL PUMP'S ACTIVATION;
- ✓ INSTANT VISUALIZATION OF TORQUE AND SPEED ADJUSTMENTS, during the operative phase;
- ✓ POSSIBILITY TO CHANGE THE OPERATION PARAMETERS and opportunity to memorize them;
- ✓ 4 PRE-PROGRAMMED FUNCTIONS by direct button.

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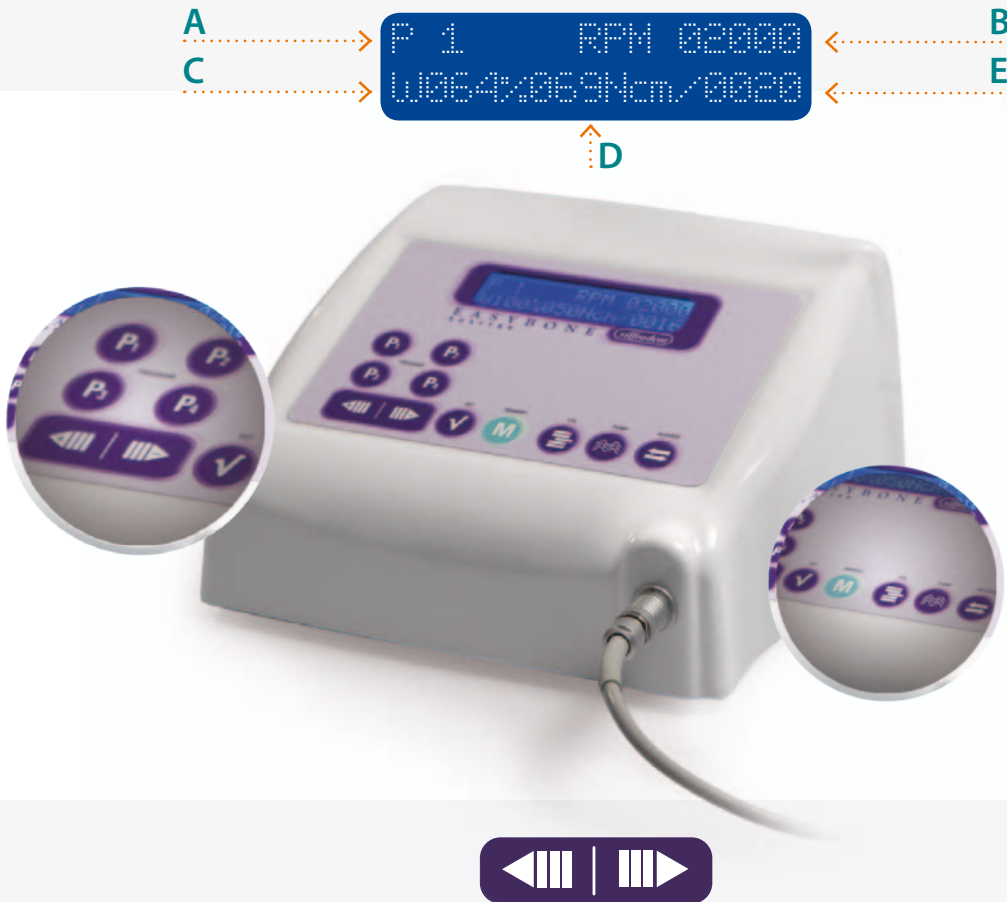
The display visualization is arranged as shown above.

The visualized parameters are specified as follows please make reference to the above figure:

- A - it indicates the **selected function** (P1, P2, P3, P4)
- B - it indicates the **speed**
- C - it indicates the peristaltic pump **power's percentage**
- D - it indicates the **torque**
- E - it indicates the **reduction/increase** of the selected parameters

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UP - By pushing this button the selected blinking parameter (speed, torque, reduction or pump) can be increased.

DOWN - By pushing this button the selected blinking parameter (speed, torque, reduction or pump) can be decreased.

SET - By pushing this button, the parameter (speed, torque, reduction or pump), can be sequentially selected. Then when the desired parameter is blinking. It can be adjusted.

MEMORY - By pushing this button, the selected blinking parameter (speed, torque, reduction or pump), can be memorized.

CAL - Insert the contra-angle with the lip. Press the button CAL and the pedal until calibration OK.

PUMP - By pushing this button, the pump status can be modified (from ON to OFF, or vice versa).

REVERSE - By pushing this button, the reverse device can be temporarily activated or deactivated. When the pedal is released, the forward mode is reintroduced.

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SPEED

Instant visualization of torque and speed adjustments, during the operative phase.

AUTO CALIBRATION

Contra-angle autocalibration upon machine's activation.

PROCEDURES

Suitable for implants, and maxillo-facial surgery and microsurgery procedures.

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CHARACTERISTIC

- ✓ **AVAILABLE FUNCTIONS:** P1-P2-P3- P4;
- ✓ **LCD GRAPHIC DISPLAY,**
(Version with 2 x 16 capitals characters) with blue backlight and white writing;
- ✓ **THE MASCHINE IS SUPPLIED WITH PRESET MODES** in terms of programs and functions.
All parameters can be modified by operator, as descibed in the instructions manual
Important: once the parameters have been modified. the pre set mode can only bema nually reacquired;
- ✓ **ELECTRONIC CONTROL** with re-programmable microprocessor;
- ✓ **POWER consumption:** 170 VA;
- ✓ **THE ENTIRE SYSTEM CONFORMS TO EC REGULATIONS** and is covered by a full guarantee all risks insurance.

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The micromotor can operate through combination to one of the following pedals, at your choice (important: pricenot included the machinery's cost)

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MULTIFUNCTIONAL PEDAL CONTROL (optional)

with reverse direction. on-off pump, start speed regulation and possibility to change functions P1/P2/P3/P4. The pedal controls Micromotor start. Suitable for operating theatres and sterile environments.

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
PNEUMATIC ON-OFF FOOTSWITCH (optional)

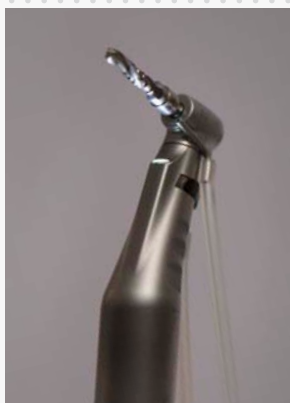suitable for operating theatres and sterile environments.

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Equipped with:

- electronic control with peristaltic pump
- high-speed motor with lighting
- multifunction pedal DI 500PV (optional)
- ON/OFF pedal SB 400P (optional)
- contrangle with light model CIS321SNLIGHT (optional)

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Equipped with:

- electronic control with peristaltic pump
- high-speed motor with optional lighting
- multifunction pedal DI 500PV (optional)
- ON/OFF pedal SB 400P (optional)
- contrangle 32:1 model CI321SN (optional)

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**OUR RESEARCH
IS JUST THE RESULT
OF YOUR
SUGGESTIONS**

A product can be developed and improved just through the experience derived from practice. Silfradent Medical Line believes in the satisfaction of user's needs. For this reason and in order to guarantee a better service, we listen to your suggestions, that you can send directly to info@silfradent.com.

Equipment for **Medical Line**

SILFRADENT
via G. di Vittorio 35/37
47018 Santa Sofia (FC) - Italy

Tel. +39 0543 970684
Fax + 39 0543 970770

www.silfradent.com
info@silfradent.com