silfradent[®]

KIT P.E.C.

Piezo Expansion Crest

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Original, innovative Technique created by Doctor Lancieri

SILFRADENT, with its 40 years of experience, manufactures products in compliance with the European regulations.

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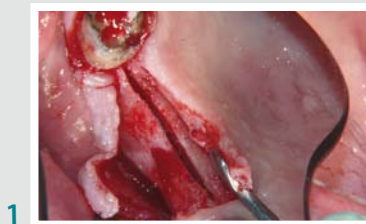
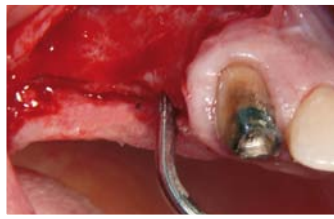
2

3

P.E.C. NON-TRAUMATIC INSERTS FOR CREST EXPANSION AND FOR THE PREPARATION OF THE IMPLANTATION TUNNEL

- Born out of a long experience developed by assiduously employing piezoelectric technology in biosurgery;
- Designed and manufactured to meet common surgery requirements, namely high control with low biologic impact;
- It reduce operation trauma to a minimum and make it possible to expand the alveolar crest with maximum simplicity, it compacts the bone and prepares the implantation tunnel, eliminating the use of rotating drills and all other systems currently in use.

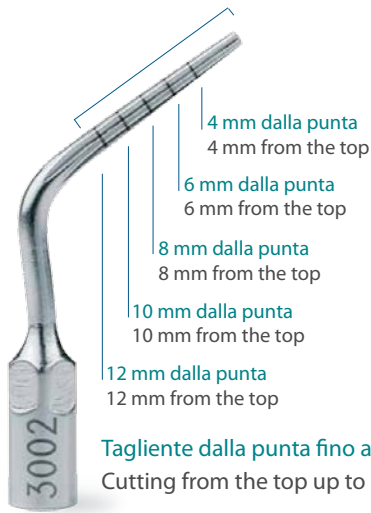
4

5

P.E.C. TECHNIQUE

- ✓ It uses bone viscoelasticity properties;
- ✓ It obtains permanent and passive bone crest plastic deformation, devoid of strains;
- ✓ Its particular design allows rapid progression with full bone preservation;
- ✓ The use of P.E.C. inserts biostimulates tissues;
- ✓ It intensifies natural regenerative properties.

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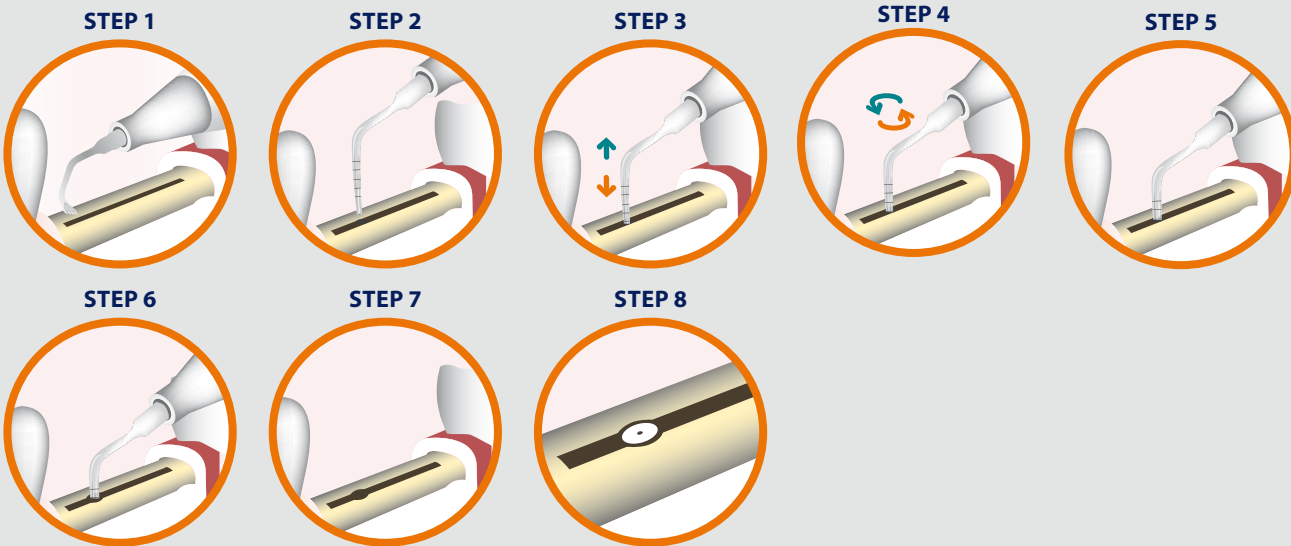
✓ **STEP 1**

Open the flap and display the ridge.
Middle crest incision using SB P0700 tip and Surgybone device.

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✓ **STEP 2 - 3 - 4 - 5 - 6 - 7 - 8**

- The first tip to use is 3001, cutting on the top.
- Make the first expansion with quick and vertical movements.
- The action has to be light reducing the pressure and lifting the tip repeatedly.
- This action has to be repeated between a notch and the one.
- It is advisable to combine a rotary action right and left (for about 30° angular) in order to obtain a clean implant site.
- Use tips in progression starting from thinnest to the thickest and end with the wanted tip.

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A METHODOLOGY FOR OSTEOIMPLANT SURGERY

Technology based on ultrasound frequencies that permits highly precise osteotomies, with a constant control of both incision depth and length.

AREA OF USE

- ✓ Sinus Lift with crestal approach
- ✓ Extraction
- ✓ Apicectomy
- ✓ Cystectomy
- ✓ Bone Harvesting
- ✓ Ridge Expansion
- ✓ Sinus Lift
- ✓ Osteoplasmy, Osteotomy
- ✓ Implant site preparation
- ✓ Tooth preparation and root planning
- ✓ Orthodontic Surgery

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OUR RESEARCH IS JUST THE RESULT OF YOUR SUGGESTIONS

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A product can be developed and improved just through the experience derived from practice. Silfradent Medical Line believes in the satisfaction of user's needs. For this reason and in order to guarantee a better service, we listen to your suggestions, that you can send directly to info@silfradent.com.

Equipment for **Medical Line**

SILFRADENT
via G. di Vittorio 35/37
47018 Santa Sofia (FC) - Italy

Tel. +39 0543 970684
Fax + 39 0543 970770

www.silfradent.com
info@silfradent.com